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PL-100

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amplifier with neckloop

PL-200

amplifier for headphone connection

Almost half of the population of Europe and North America now owns a mobile phone. To ensure that you too can stay in touch when on the move, we're offering the solution for both hearing aid wearers (B.T.E. and I.T.E.) and non hearing aid wearers. The mobile phone amplifiers are simply connected to your mobile phone using an adapter which is readily available in the shops. The battery operated amplifier adjusts the volume of the incoming call using a variable volume control. The amplifier is powered by 3 hearing aid batteries (type 675 = zinc-air for heavy use or LR44 = alkali-manganese).

A variable attachment clip is included with both models.

Technical data

Dimensions:

H x W x D 53 x 48 x 26 mm, colour slate grey

Weight 41 g

Acoustic amplification approx. + 25 dB \pm 10 %

Inductive amplification approx. + 35 dB \pm 10 %

With power on/off lamp

PL-100 for inductive amplification

The amplifier produces an induction loop signal that can be received by a hearing aid with a built-in telecoil. This function is activated by setting the hearing aid to position »T« or »MT«. The induction loop signal can be increased by turning the volume control until a comfortable volume level is heard.

PL-200 for headphones or direct connection

The amplified sound signal is output using a pair of lightweight headphones (available in shops, 8 to 32 Ohm impedance, e.g. stethoscope headsets order no. A-4902-0 or lightweight headphones order no. A-4905-0). Furthermore, the PL-200 can be directly connected to the hearing aid by means of an audio cable with audio connector (order no. A-4940-0 or order no. A-4950-0).

Order no:

A-4540-0 PL-100: Mobile phone amplifier with neckloop.

A-4550-0 PL-200: Mobile phone amplifier for headphone or direct connection.

Performance features

- Acoustic or inductive amplifier.
- Small, lightweight and portable.
- Suitable for many modern types of mobile phone.
- Variable volume control.
- Built in microphone.
- Power supply from 3 hearing aid batteries (type LR44 supplied).
- 12 months guarantee.

Adapter

The PL-100 and PL-200 each have a 2.5 mm jack socket for use with Hands Free ear pieces. The hearing amplifiers can be connected to various types of mobile phone using headset adapters specifically for mobile phones (not supplied). Please make sure the headset adapter has a 2.5 mm jack plug for use with Hands Free ear pieces.

Visit our website at www.humantechnik.com for a list of adapters which can be used.

We would be pleased to give you detailed advice.