LPU-1

D	Bedienungsanleitung Ringschleifen-Empfänger	Seite 2
GB	Operating Instructions Induction Loop Receiver	Page 6
F	Mode d'emploi Récepteur de boucle magnétique	Page 10
NL	Gebruiksaanwijzing Ringleidingontvanger	Pagina 14
I	Istruzioni per l'uso Ricevitore a loop circolare	Pagina 18
E	Instrucciones de servicio Receptor en bucle	Página 22

LPU-1
Ringschleifen-Empfänger

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
LPU-1
Induction Loop Receiver

Special accessories:

- A-4275-0 Induction loop receiver with 1 battery (without charging station)
- A-4973-0 Single charger for receiver
- A-4974-0 Multiple charger for 12 accus
- A-4955-0 Plastic shell for 5 stethoscope receivers

2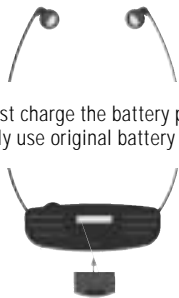


When you first charge the battery packs let them charge for at least 14 hours. Only use original battery packs.

3

Make sure that the stethoscope receiver is worn correctly


4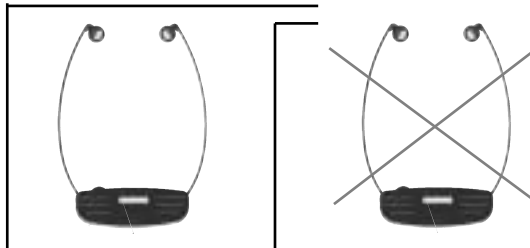


Reception range:

The induction loop receiver can receive signals which are transmitted by induction loop systems. This corresponds to a hearing aid in the »T« position. Ensure that the induction loop receiver is operated within an installed induction loop. The induction loop receiver does not work outside this range.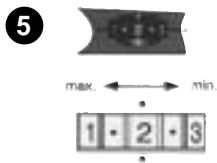

5 **Volume adjustment of the receiver.**
There is no ON/OFF switch.

The receiver automatically switches on when worn.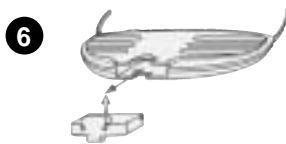

6 If the sound quality of the receiver becomes noisy and distorted the battery pack should be exchanged by a loaded one.


Warning: do not force the battery pack into the charger or receiver. It fits easily when correctly inserted.

7 **Cleaning the hygienic covers**

The hygienic covers contained in the standard delivery package should be cleaned when they become dirty.

To do this use the Hygienic-SET offered as a special accessory (order no. A-4982-0).

8 Please note the following information about your LPU-1:

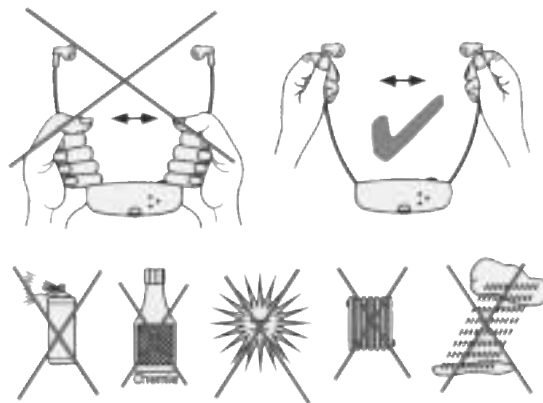If the system needs cleaning ONLY use a soft damp cloth.
If the foam ear pieces become damaged they should be replaced.

9 Technical data:

Frequency response	70 - 5400 Hz
Distortion	< 1 %
Noise level	typ. 60 dB
Battery charging time	approx. 14 h
Operating time	approx. 6 h
Maximum volume SPL	approx. 120 dBA
Weight	47 g

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Humantechnik GHL AG
ELECTRONIQUE S.A.R.L**